


Patient Instruction Sheet

trackit T4A

Upon arrival at home, after having the equipment and electrodes fitted to you, please observe the following instructions to help ensure a successful recording:

DO:


- ✓ Keep the equipment in its bag to protect it from ingress of water.
- ✓ Ensure that the bag and straps are worn over clothing to prevent any possibility of skin irritation.
- ✓ Take care to avoid entanglement of cables or straps. Entangled cables or straps could result in personal injury or strangulation.
- ✓ Press the event button (one single quick press) if you feel anything unusual 
- ✓ Contact the number, provided above, if the EEG electrodes become loose or detached, or if you experience any problems with the equipment.

Environmental operating conditions

Temperature	+5°C – +40°C (+41°F to +104°F)
Humidity	15% – 90%
Protection class	IP22, when contained in the bag. IP22 Explanation <i>1st digit (2): Protected against foreign objects greater than 12.5mm. 2nd digit (2): Protected against dripping water when the equipment is tilted up to 15°.</i>

For help, call:

DON'T:

- ✗ Do not tamper with the equipment or attempt any modification.
- ✗ The equipment requires no patient intervention during the recording. Once the recording has been set up, do not touch / tamper with the laptop during a recording.
- ✗ Do not remove the battery packs from the amplifier. Contact the number, provided above, if the battery packs need replacing. The battery capacity is shown on the LCD display ( symbol).
- ✗ Do not use mobile phones in close proximity to the equipment.
- ✗ Do not chew gum or other sweets as this produces artefact signals.
- ✗ Do not bath or shower.
- ✗ Do not subject the equipment to heat or moisture. See Environmental operating conditions.
- ✗ Do not allow pets or children to interfere with the equipment or the cables.



CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

51318-006 - T4A EEG Amplifier Patient Instruction Sheet Iss 1.4 Part No. 1608