

# Trackit t4A EEG Amplifier



## *Patient Instruction Sheet*

**For help, call:**

Upon arrival at home, after having the equipment and electrodes fitted to you, please observe the following instructions to help ensure a successful recording:

### **Do**

- ✓ Keep the equipment in its bag to protect it from ingress of water.
- ✓ Ensure that the bag and straps are worn over clothing to prevent any possibility of skin irritation.
- ✓ Take care to avoid entanglement of cables or straps. Entangled cables or straps could result in injury.
- ✓ Press the event button (one single quick press) if you feel anything unusual (⊙)

### **Don't**

- ✗ Do not tamper with the equipment or attempt any modification. The equipment requires no patient intervention during the recording.
- ✗ Do not remove the battery packs from the amplifier. Contact the number, provided above, if the battery packs need replacing.
- ✗ Do not use mobile phones in close proximity to the equipment.
- ✗ Do not chew gum or other sweets as this produces artefact signals.
- ✗ Do not bath or shower.
- ✗ Do not subject the equipment to heat or moisture.
- ✗ Do not allow pets or children to interfere with the equipment or the cables.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

CE 0086