

## Patient Instruction Sheet


**For help, call:**

Upon arrival at home, after having the equipment and electrodes fitted to you, please observe the following instructions to help ensure a successful recording:

### **Do**

- ✓ Keep the equipment in its bag to protect it from ingress of water.
- ✓ Ensure that the bag and straps are worn over clothing to prevent any possibility of skin irritation.
- ✓ Take care to avoid entanglement of cables or straps. Entangled cables or straps could result in personal injury or strangulation.
- ✓ Press the event button (one single quick press) if you feel anything unusual (⊙)
- ✓ Contact the number, provided above, if you experience any problems with the equipment.

### **Don't**

- ✗ Do not tamper with the equipment or attempt any modification. The equipment requires no patient intervention during the recording.
- ✗ Do not remove the battery packs from the amplifier. Contact the number, provided above, if the battery packs need replacing. The battery capacity is displayed on the front panel (  symbol).
- ✗ Do not use mobile phones in close proximity to the equipment.
- ✗ Do not chew gum or other sweets as this produces artefact signals.
- ✗ Do not bath or shower.
- ✗ Do not subject the equipment to heat or moisture. See Environmental operating conditions, below.
- ✗ Do not allow pets or children to interfere with the equipment or the cables.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

<b>Environmental operating conditions</b>	
Temperature	+5°C – +40°C
Humidity	15% – 90%
Protection class	IP22, when contained in the bag.
<b>IP22 Explanation</b>	
1 <sup>st</sup> digit (2): Protected against foreign objects > 12.5mm.	
2 <sup>nd</sup> digit (2): Protected against dripping water when the equipment is tilted up to 15°.	

 0086