

REVIEW WEBPAGE:


Event List

The **Event List** is located at the bottom right-hand corner of the **Review Webpage**. To pop out the Event List, click the **Popout** icon.



The **Event List** can be sorted by clicking the column titles. The sorting is on a three-click cycle by clicking anywhere in the **headers** at the top: For example, if you click the column title **"Name"**:

- First click – Alphabetized A-Z
- Second click – Alphabetized Z-A
- Third click – Original View

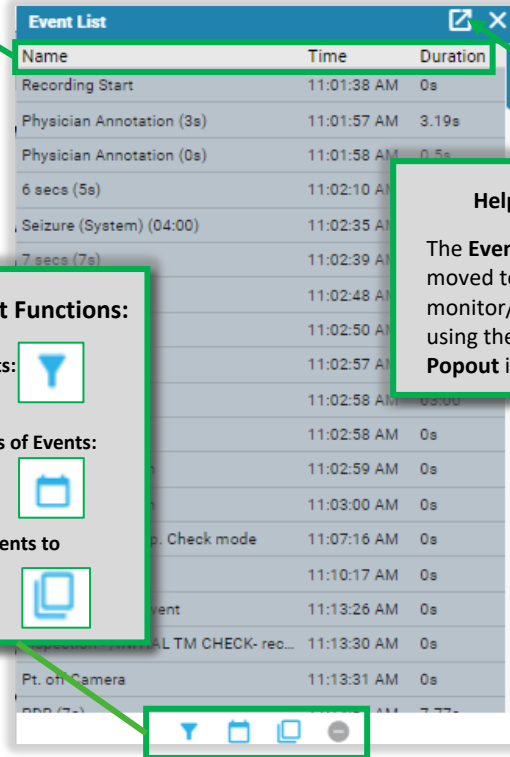

Name	Time	Duration
Recording Start	11:01:38 AM	0s
Physician Annotation (3s)	11:01:57 AM	3.19s
Physician Annotation (0s)	11:01:58 AM	0.5s
6 secs (5s)	11:02:10 AM	5.66s
Seizure (System) (04:00)	11:02:35 AM	04.00s
7 secs (7s)	11:02:39 AM	7.04s
S/W Duration (7s)	11:02:48 AM	7s
PDR (4s)	11:02:50 AM	4.86s
Patient - Test	11:02:57 AM	0s
AWAKE (03:00)	11:02:58 AM	03:00s
Patient - Test	11:02:59 AM	0s
Patient - Test push	11:03:00 AM	0s
Patient - Test push	11:03:00 AM	0s
ImpChkMode - Imp...	11:07:16 AM	0s
AA (blinking)	11:10:17 AM	0s
Impedance Test Ev...	11:13:26 AM	0s
Inspection - /INTL	11:13:30 AM	0s
Pt. off Camera	11:13:31 AM	0s

Helpful Tip:

The **Event List** can be moved to a separate monitor/screen by using the **secondary Popout** icon.

Event List Functions:

Filter Events:



Show Dates of Events:



Copy All Events to Clipboard:

