


PATIENT VISITS LIST:

How to Search and Filter the Patient Visit List

All Patient Exams can be found in the Patient Visit List. **Search Parameters** may be used to find a specific patient/exam.

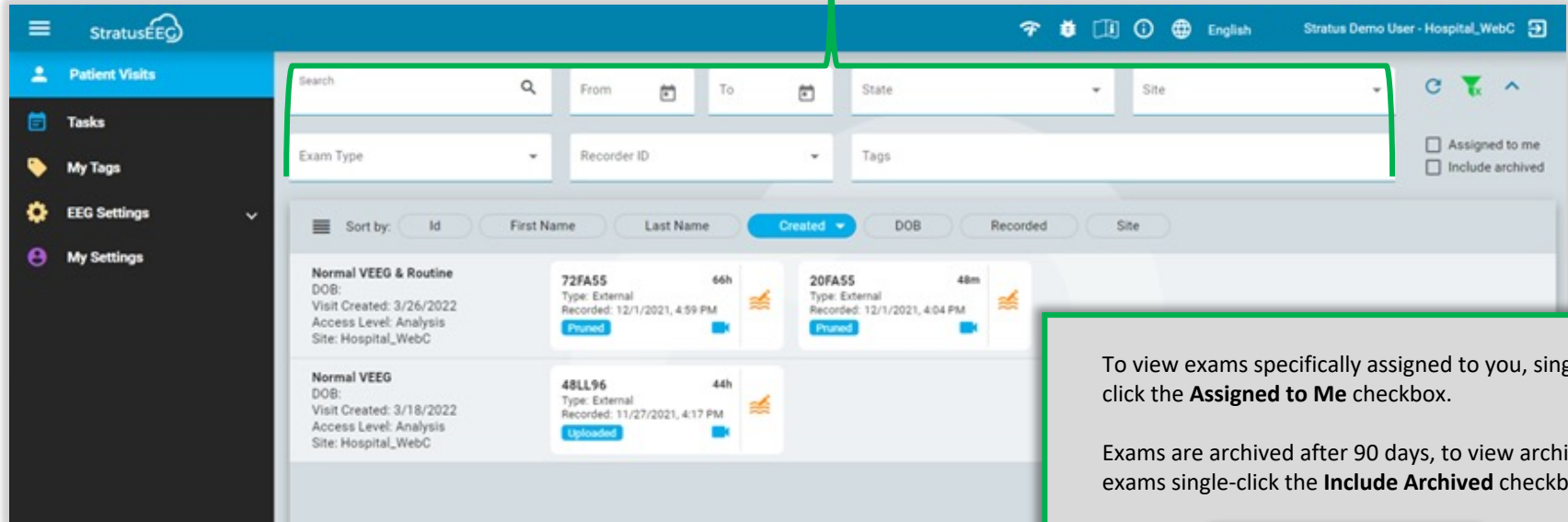

To view exams specifically assigned to you, single-click the **Assigned to Me** checkbox.

Exams are archived after 90 days, to view archived exams single-click the **Include Archived** checkbox.

- Assigned to me
- Include archived

PLEASE NOTE: Once any filters have been activated you will notice the **Filter Icon** will appear **green**, to clear filters single-click the **green Filter Icon**

