


# PATIENT VISITS LIST:

## Opening an Exam from the Patient Visits List or the Patient Folder

### Patient Visits List:

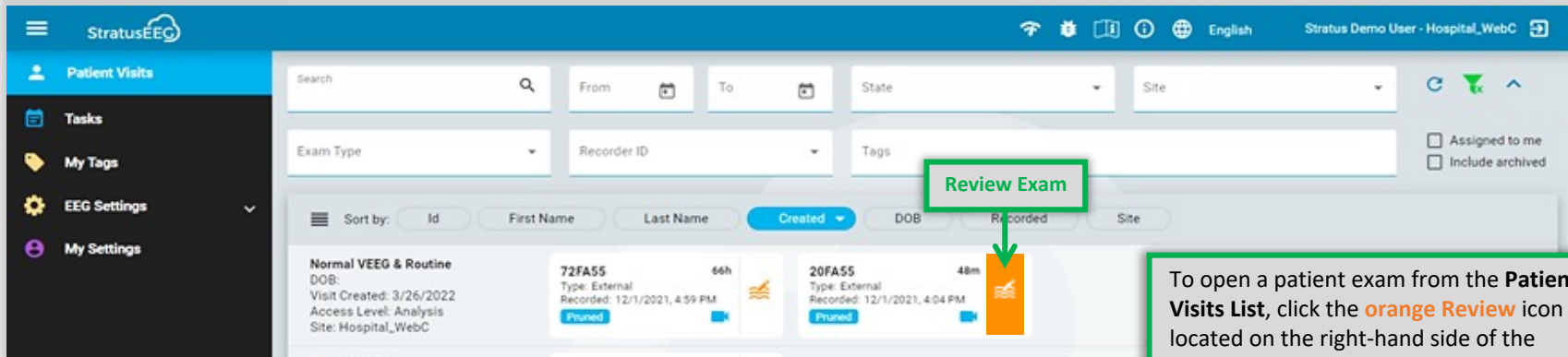


Normal VEEG & Routine  
DOB: [redacted]  
Visit Created: 3/26/2022  
Access Level: Analysis  
Site: Hospital\_WebC

Exam ID	Type	Duration	Recorded	Pruned	Review Exam
72FA55	External	66h	12/1/2021, 4:59 PM	Pruned	Review Exam
20FA55	External	48m	12/1/2021, 4:04 PM	Pruned	

To open a patient exam from the **Patient Visits List**, click the **orange Review** icon located on the right-hand side of the preferred **Exam Box**.

### Patient Folder:

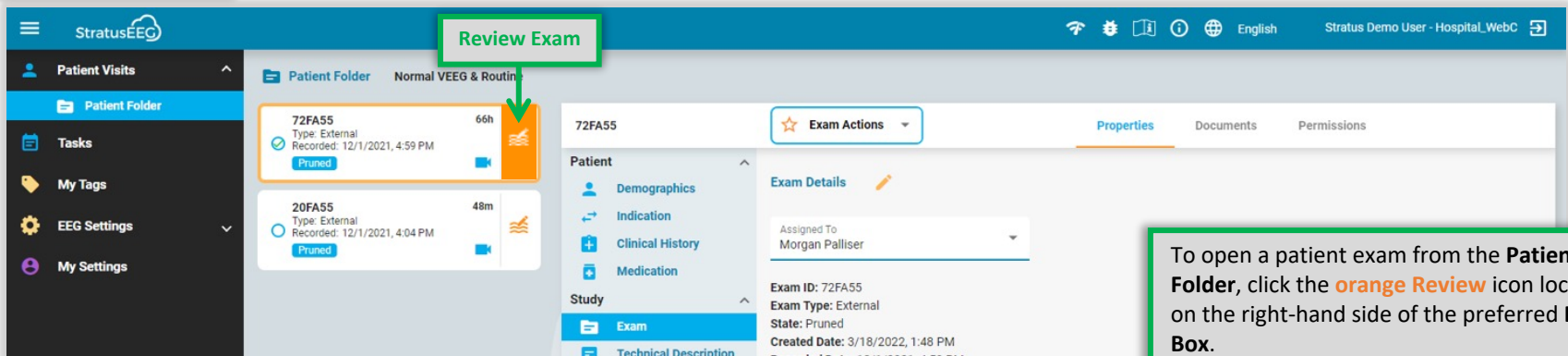

Normal VEEG & Routine

72FA55	Type: External	66h	Recorded: 12/1/2021, 4:59 PM	Pruned	Review Exam
20FA55	Type: External	48m	Recorded: 12/1/2021, 4:04 PM	Pruned	

Exam Actions: [star icon] [dropdown arrow]

Properties: Documents, Permissions

Patient: Demographics, Indication, Clinical History, Medication

Study: Exam, Technical Description

Exam Details: Assigned To: Morgan Palliser

Exam ID: 72FA55  
Exam Type: External  
State: Pruned  
Created Date: 3/18/2022, 1:48 PM  
Recorded Date: 12/1/2021, 4:59 PM

To open a patient exam from the **Patient Folder**, click the **orange Review** icon located on the right-hand side of the preferred **Exam Box**.