


PATIENT VISITS LIST:

Information Included in a Patient Card

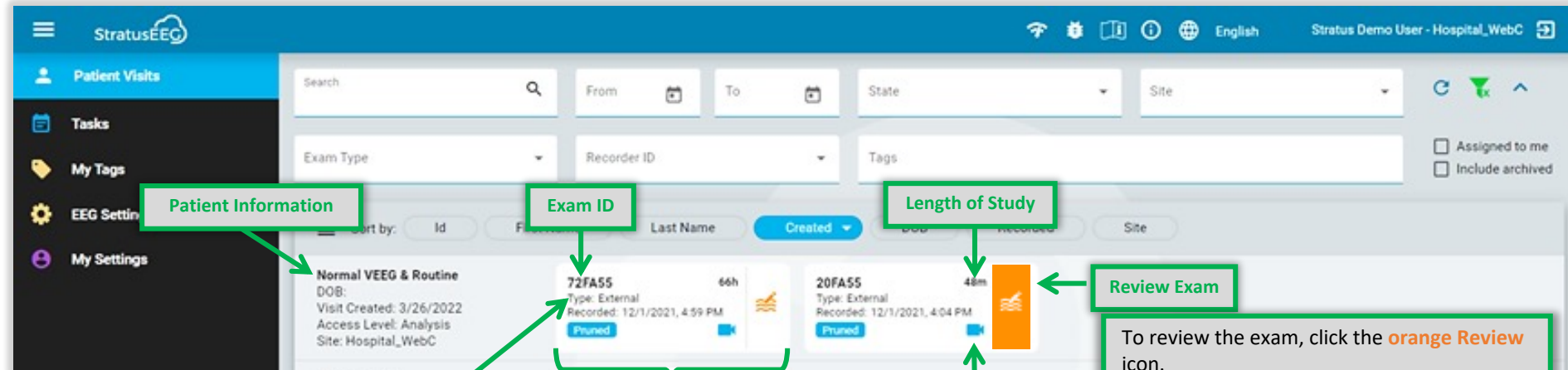

The screenshot shows the StratusEEG Patient Visits interface. A sidebar on the left contains navigation options: Patient Visits, Tasks, My Tags, EEG Settings, and My Settings. The main area features a search bar and filters for Exam Type, Recorder ID, and Tags. A table lists patient visits with columns for Id, Exam ID, Last Name, Created, Length of Study, and Site. Two exam cards are visible: one for 'Normal VEEG & Routine' (ID: 72FA55, 66h) and another for 'Normal VEEG & Routine' (ID: 20FA55, 48m). Callouts point to the Patient Information section, Exam ID, Length of Study, and an orange 'Review Exam' icon.

Exam Type


Each Exam Box refers to a single exam

Video Included

Review Exam

To review the exam, click the orange Review icon.

To open the **Patient Folder** and view exam properties, hover your cursor over the preferred Patient Card and click anywhere within the **blue-highlighted area**.

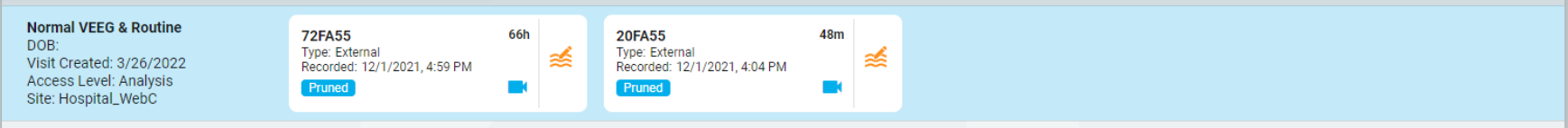

This block shows a detailed view of a patient card. On the left, it displays patient information: 'Normal VEEG & Routine', 'DOB:', 'Visit Created: 3/26/2022', 'Access Level: Analysis', and 'Site: Hospital_WebC'. To the right, there are two exam cards. The first card shows '72FA55', 'Type: External', 'Recorded: 12/1/2021, 4:59 PM', a 'Pruned' button, a video icon, and a duration of '66h'. The second card shows '20FA55', 'Type: External', 'Recorded: 12/1/2021, 4:04 PM', a 'Pruned' button, a video icon, and a duration of '48m'.