


REVIEW WEBPAGE:

Standard Features for Reviewing EEG Data and Video

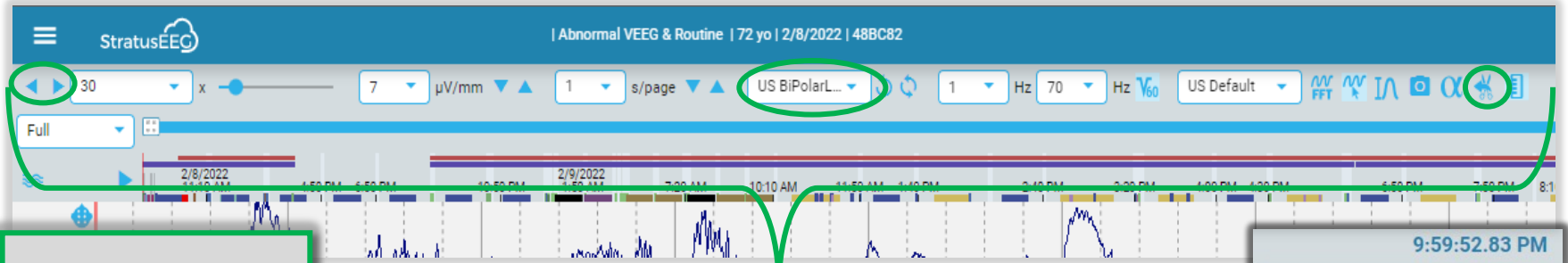


Standard Features:

Play Forward/Backward Buttons:



Montage Dropdown Menu:

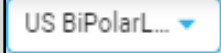

Prune/Undo Prune Icon:



Prune:

Click the **Prune** button to hide the pruned portions of the exam



Undo Prune:

Click the **Undo Prune** button to unhide the pruned portions of the exam

The **Review Toolbar** is located at the top of the Review webpage. The user can customize their review display by making preferred changes to their toolbar settings.

The **Video Review Panel** can be found on the right-hand side of the Review Webpage

- To hide the Panel for more EEG screen visibility, click the **Hide Panel** button
- To pop out the Video, Amplitude, and Event List Windows, click the pop-out icons

