

REVIEW WEBPAGE HAMBURGER:


How to Complete Screen Calibration

1 Navigate to the Review page and click the **Hamburger** located at the top left-hand side of the webpage.

2 Click **Screen Calibration**.



3 Follow the instructions by matching the rulers to a real ruler (metric) by holding it up to the screen. Drag the ends of the screen rulers to line up with the real ruler (10 cm).

Click **Save** when complete.

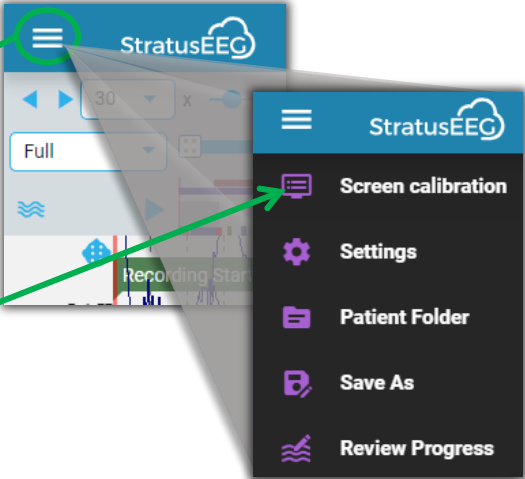

PLEASE NOTE:

The screen needs to be calibrated each time an exam is opened on a new monitor for the ruler to show an accurate display of 5 centimeters. Once this is done, the position of the ruler changes to accurately reflect the amplitude of the EEG.

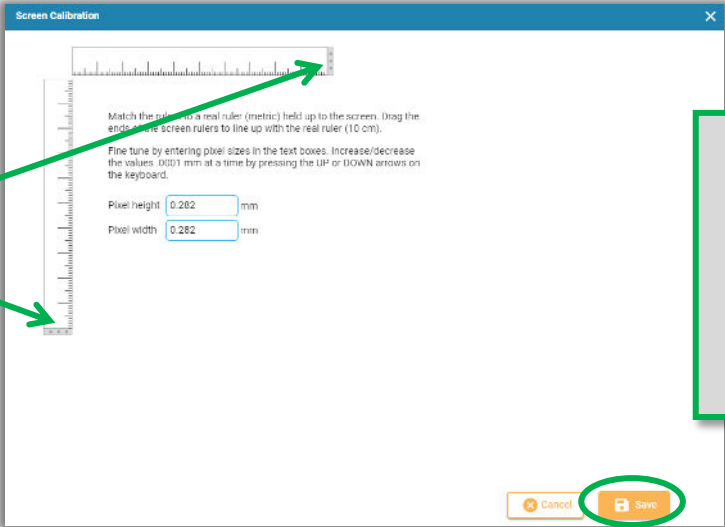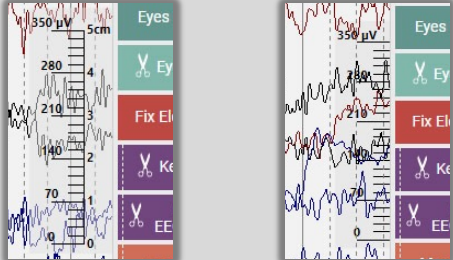

Helpful Hint:

After adjusting the screen rulers, take note of the Pixel height & Pixel width if you need to calibrate the same screen in the future.