


New Features in Trackit Plus v2.8.4.7


ADDITIONAL PROGRAM OPTIONS

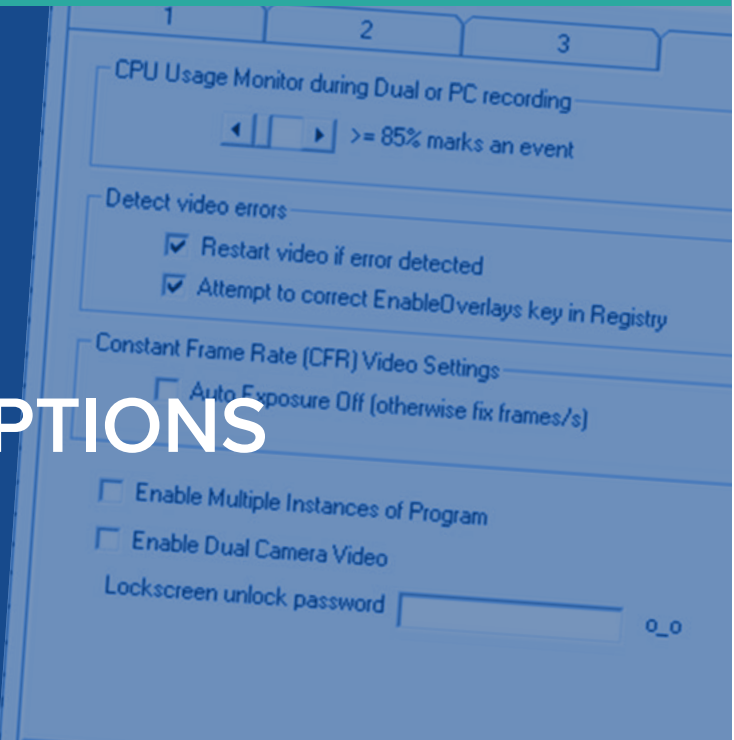

Additional Program Options

CPU Usage Monitor during Dual or PC recording


This setting allows a configurable CPU usage limit to mark an event in the recording. Such an event is useful for diagnosis when an abnormally high CPU load causes issues with the recording

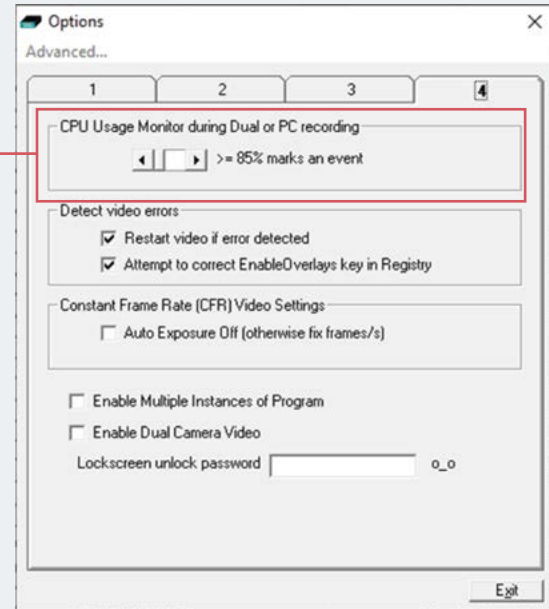

Additional Program Options (continued)

Detect video errors


During video recording, the program will attempt to detect video errors and automatically recover.

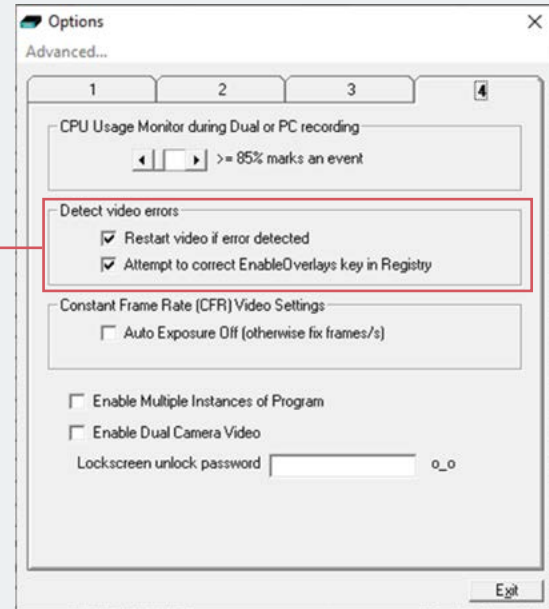

Additional Program Options (continued)

Constant Frame Rate (CFR) Video Settings


This setting is for the video camera and determines if Auto Exposure is used or not. When On, the camera can automatically adjust its frame rate to compensate for low light levels. This can effect video/EEG synchronisation, so an option is provided to disable this and instead fix the frame rate.

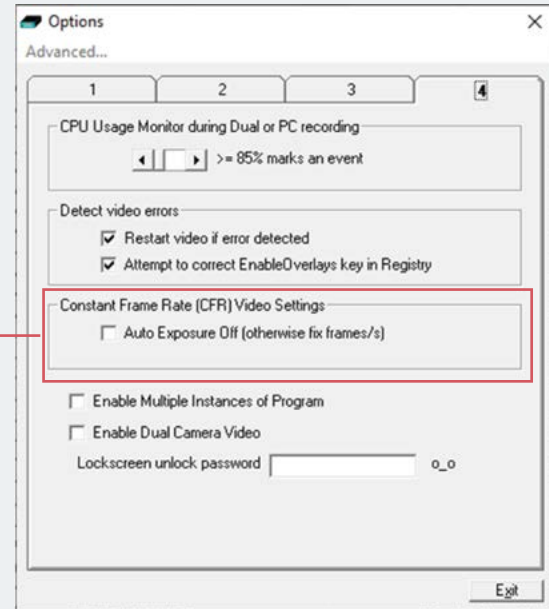

Additional Program Options (continued)

Enable Multiple Instances of the Program


If enabled, multiple instances of the program are enabled. This is not normally required, since the program supports simultaneous review in acquire.

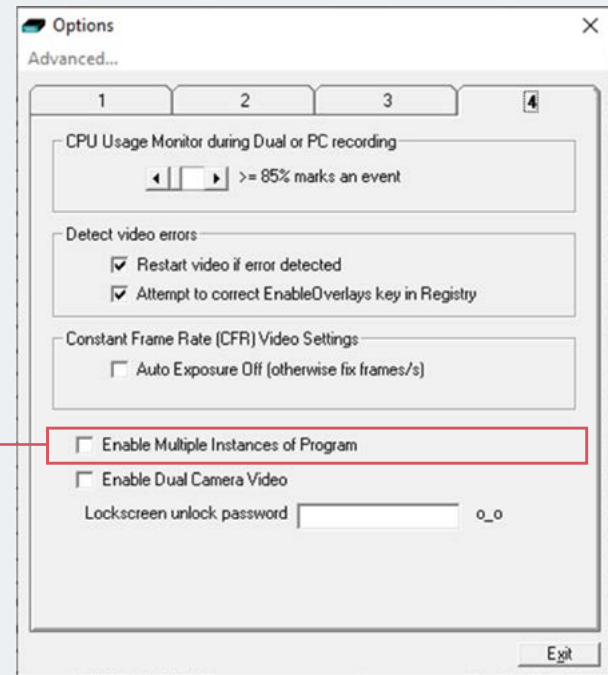

Additional Program Options (continued)

Enable Dual Camera Video


When selected, allows dual video camera support. Otherwise, only a single camera is allowed.

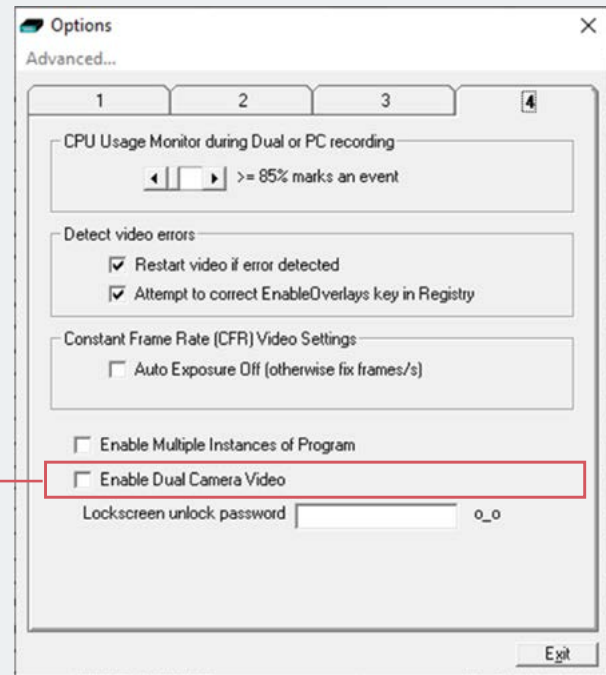

Additional Program Options (continued)

Lockscreen unlock password

Allows setting a custom Lockscreen unlock password. Note that there is a default hidden password of 'lifelinesXX', where XX is the numeric current day of the month, e.g. 3 or 24 etc. This password always works.

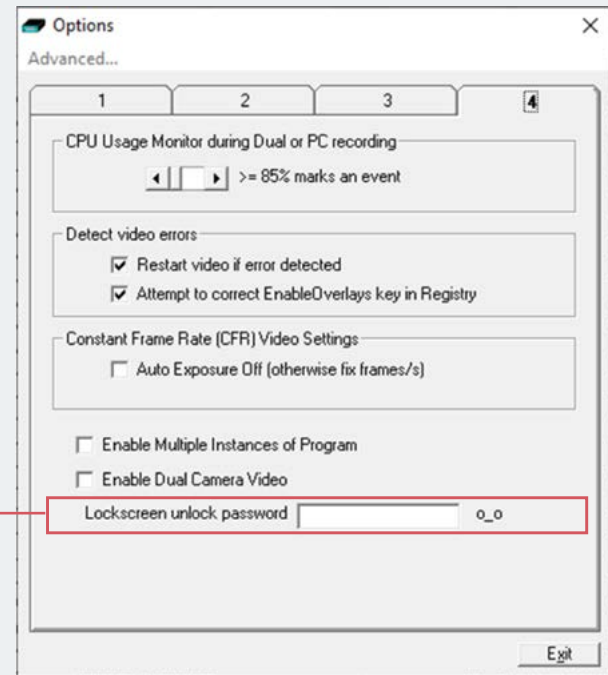



For additional assistance

Tel: 866-889-6505

Email: support@lifelinesneuro.com