

## New Features in Trackit Plus v2.8.4.7

### NEW EVENTS LOGGED



## New Events Logged

There are several automatic events recorded with a study. These include:

### Amp Battery % at start of study.

The actual % capacity of the battery is read from the amplifier and added as an event at the start of the recording.



### Event for lock screen enter/Event for lock screen exit.

The period between the Enter and Exit Lockscreen events is the time spent in the Lockscreen and the next Enter event gives the time out of the Lockscreen.



### Event for Review enter/Event for Review exit.

The period between the Enter and Exit Review events is the time spent in Review.





## For additional assistance

Tel: 866-889-6505

Email: [support@lifelinesneuro.com](mailto:support@lifelinesneuro.com)